

# Posterior Occlusion Method with Softened Paraffin Wax P O M


If a patient told that the denture currently in use is "come off", "cannot bite", "hurt", why not suspect occlusion?

① please ask a patient, do inspection, and palpation.

Examination

Diagnosis

①

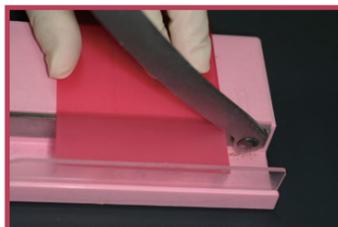

② Please cut paraffin wax into 24mm size for checking bite.



③ Softening paraffin wax with fire.



④ Folding the paraffin wax into about 7mm to 10mm two times, and make the blocks with three times thickness.



⑤ put it on posterior and cut it with finger.

Bite registration

=

**POM**  
(Posterior Occlusion Method with Softened Paraffin Wax)

② · ③ · ④ · ⑤ · ⑥ · ⑦




⑥ do the same on opposite side.

⑦ take bite registration. Deliver the denture into a patient's mouth. And please take the bite registration at high occlusal position without touching occlusion in posterior. while you take the bite registration, you talk to patients "please bite gently. Do not chew. open and close gently."

**Q** What is Posterior Occlusion Method with softened paraffin wax?

**A** This method makes the denture which doesn't work properly drastically improved.

## flow chart

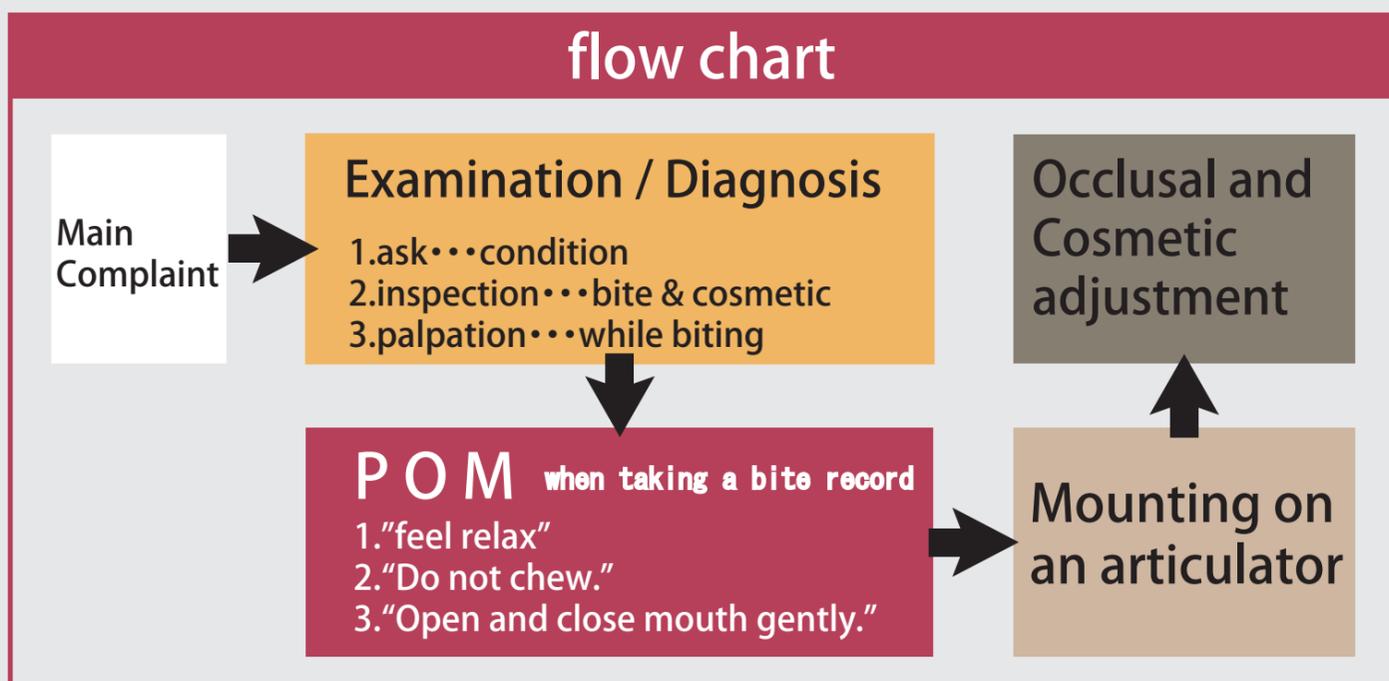

Mounting on an articulator ⑧ · ⑨ · ⑩ · ⑪



⑧ take a impression and make mold with special mold material called "kisantano" then mount it on an articulator with putting the wax between teeth.



⑨ Keep the bite at high position in posterior with two pieces thickness of wax.(about 3mm)



⑩ · ⑪ Once take the incisal pin out, it happens interceptive occlusal contact in anterior, it doesn't show occlusal contact in posterior.



⑮ the patient didn't feel pressure from below, the denture is totally improved.



⑭ The patient couldn't keep a stable bite position before adjustment. But after adjustment, the patients can find a right bite position without being conscious.



⑬ check bite with articulating paper.



⑫ adjust with grinding artificial teeth on an articulator.

Final delivery

⑭ · ⑮

Occlusal and cosmetic adjustment

⑫ · ⑬